
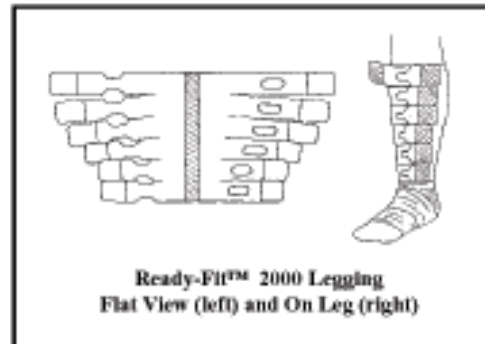


# CircAid® Systems


## Ready-Fit™ 2000 and Ready-Fit™ Classic Leggings Measuring and Sizing Chart

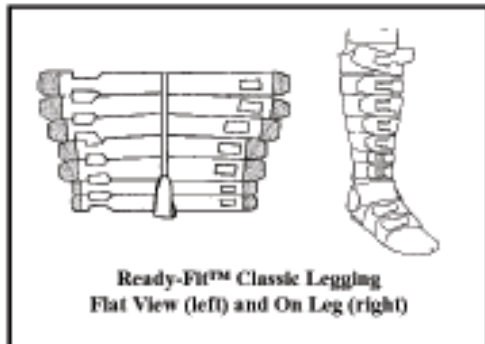

### 1. Length Measurement:

1/2" below knee crease to 1/2" above center of ankle bone.

enter here in  inches  cm

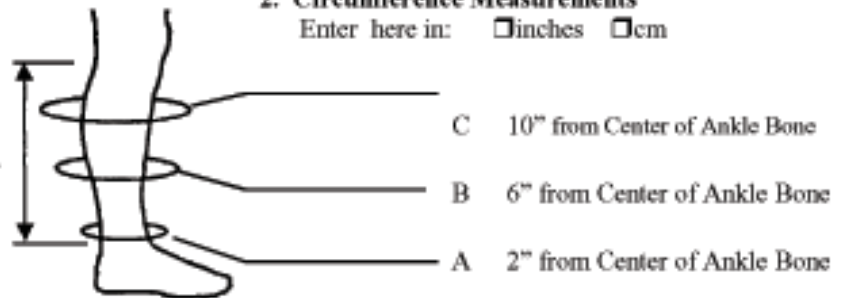

### 3. Other Instructions:

Be sure measurements are accurate. When on the borderline between sizes, consider the amount of edema present and the anticipated reduction. If significant edema is present and reduction is anticipated, select the smaller size; if edema is minimal select the larger size. ALWAYS USE THE UNDERSOCK OR STOCKINETTE WHEN TESTING A SIZE.

Standard length is 14".

Circumference Measurement	Small		Medium		Large		XLarge		XXLarge	
	<b>A</b> (2" from ankle)	6"-9"	15-23cm	8"-11"	20-28cm	10"-13"	25-33cm	12"-15"	30-38cm	13"-16"
<b>B</b> (6" from ankle)	8"-11"	20-28cm	10"-13"	25-33cm	12"-15"	30-38cm	14"-18"	36-46cm	17"-21"	43-53cm
<b>C</b> (10" from ankle)	12"-15"	30-38cm	14"-17"	36-43cm	16"-19"	41-48cm	18"-22"	46-56cm	21"-25"	53-64cm

**CircPlus II Ankle-Foot Wrap:** All CircAid Leggings come with an Ankle-Foot Wrap, fitting up to an arch circumference of 15", men's shoe size 12, and a women's shoe size 13. For abnormally large or unusually shaped ankles, please call CircAid Medical for sizing information.